

HydroGen



Effective removal of devitalised tissue

HydroGen hydrogel dressing range

Product Description

An amorphous hydrogel presented in two presentations:

- A free standing cross-linked gel
- A Gel-sheet with barrier film top

Mode of Action

Positioned in direct contact with the wound bed.

Hydrogen amorphous Hydrogel will donate fluid to the wound to soften and lift necrotic and sloughy tissue.

It will also absorb any excess exudate present in the wound.

Helps to stimulate wound healing, by removing non-viable tissue, thus helping achieve granulation quicker.

Benefits

Both absorbs and donates, depending on the wound conditions.

Sheet and Island provide a simple one hand application, without the need for a secondary dressing.

High structural strength.

Gelling actions provide cooling, soothing effect for enhanced patient comfort.

Easy to apply.

Indications

- Venous ulcers
- Diabetic ulcers
- Pressure Ulcers
- Trauma wounds
- Burn wounds

Contra-Indications

None.

Precautions

HydroGen is sterile only if the package is dry, unopened and undamaged.

Discard product if the package seal is opened or contaminated.

The product must be used prior to the expiration date.

Sizes and Codes

Size	Carton Quantity	Order code
Amorphous Gel		
15 gram	Carton of 12	HG0015
Gel-sheet		
9.5cm x5.5cm	Carton of 12	HS0402
12cm x 8cm	Carton of 12	HS0503

Island gel-sheet

7.5cm x 4.5cm	Carton of 10	Hi0302
10cm x 10cm	Carton of 10	Hi0404

